


# Important information you need to know before injecting Mounjaro.

Read this instructions for use and the package leaflet before using your Mounjaro pre-filled pen (pen) and each time you get a new pen. There may be new information. This information does not take the place of talking to your doctor, pharmacist or nurse about your medical condition or treatment.

Talk to your doctor, pharmacist or nurse about how to inject Mounjaro the right way.

- Mounjaro is a single-dose pre-filled pen.
- The pen has a hidden needle which will automatically insert into your skin when the injection button is pressed. The pen will retract the needle when the injection is completed.
- Mounjaro is used 1 time each week.
- Inject under the skin (subcutaneously) only.
- You or another person can inject into your stomach (abdomen), upper leg (thigh) or upper arm.
- You may need help from someone else if you want to inject in your upper arm.

## Guide to parts

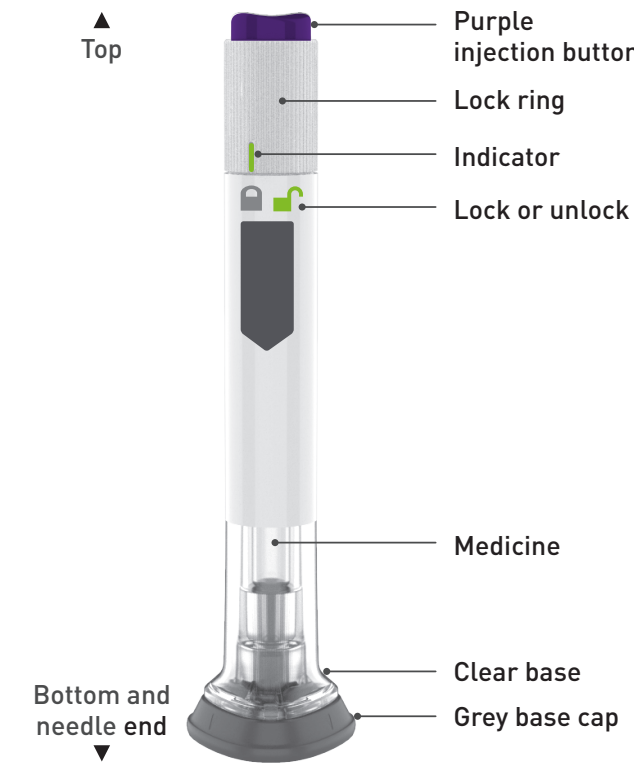

2 of 13

## Preparing to inject Mounjaro

**Remove** the pen from the refrigerator. Leave the grey base cap on until you are ready to inject.


**Check** the pen label to make sure you have the right medicine and dose, and that it has not expired.

**Inspect** the pen to make sure that it is not damaged.

Make sure the medicine is:

- not frozen
- not cloudy
- colourless to slightly yellow
- does not have particles


**Wash** your hands.

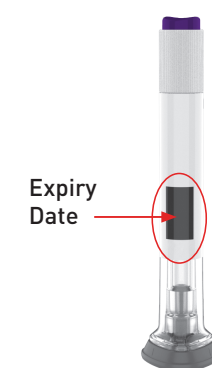

3 of 13

## Choose your injection site

Your doctor, pharmacist or nurse can help you choose the injection site that is best for you.

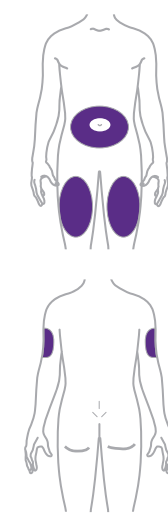

You or another person can inject the medicine in your stomach (abdomen) or thigh.


Another person should give you the injection in the back of your upper arm.

Change (rotate) your injection site each week.

You may use the same area of your body, but be sure to choose a different injection site in that area.

4 of 13

## Step 1 Pull off the grey base cap

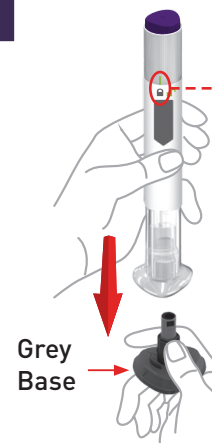

Make sure the pen is **locked**.

**Do not** unlock the pen until you place the clear base on your skin and are ready to inject.

**Clear Base**


**Pull** the grey base cap straight off and throw it away.

**Do not** put the grey base cap back on – this could damage the needle.


**Do not** touch the needle.

5 of 13

## Step 2 Place clear base on skin, then unlock

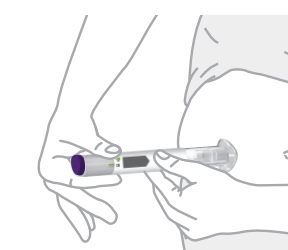

**Place** the clear base flat against your skin at the injection site.

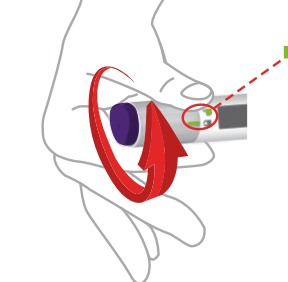

**Unlock** by turning the lock ring.

6 of 13

## Step 3 Press and hold up to 10 seconds



**Press and hold** the purple injection button.

**Listen** for:

- First click = injection started
- Second click = injection completed



You will know your injection is complete when the grey plunger is visible.

After your injection, place the used pen in a sharps container.

7 of 13

CONTINUED ON BACK

## Disposal of your used pen

- Throw away (dispose of) the pen in a sharps container or as directed by your doctor, pharmacist or nurse. **Do not** throw away (dispose of) pens in your household waste.
- Do not recycle your used sharps disposal container.
- Ask your doctor, pharmacist or nurse about how to dispose of medicines you no longer use.



8 of 13

## Storage and handling

- For storage instructions refer to section 5 of the patient information leaflet.
- The pen has glass parts. Handle it carefully. If you drop the pen on a hard surface, **do not** use it. Use a new pen for your injection.

9 of 13

## Commonly asked questions

### What if I see air bubbles in my pen?

Air bubbles are normal.

### What if my pen is not at room temperature?

It is not necessary to warm the pen to room temperature.

### What if I unlock the pen and press the purple injection button before pulling off the grey base cap?

**Do not** remove the grey base cap. Throw away the pen and get a new pen.

### What if there is a drop of liquid on the tip of the needle when I remove the grey base cap?

A drop of liquid on the tip of the needle is normal. **Do not** touch the needle.

### Do I need to hold the injection button down until the injection is complete?

This is not necessary, but it may help you keep the pen steady against your skin.

10 of 13

### I heard more than 2 clicks during my injection — 2 loud clicks and 1 soft one. Did I get my complete injection?

Some people may hear a soft click right before the second loud click. That is the normal operation of the pen. **Do not** remove the pen from your skin until you hear the second loud click.

### I am not sure if my pen worked the right way.



Check to see if you have received your dose. Your dose was delivered the right way if the grey plunger is visible. Also, see **Step 3** of the instructions.

If you do not see the grey plunger, contact your local Lilly affiliate for further instructions. Until then, store your pen safely to avoid an accidental needle injury.

### What if there is a drop of liquid or blood on my skin after my injection?

This is normal. Press a cotton ball or gauze over the injection site. **Do not** rub the injection site.

11 of 13

## Other information

- If you have vision problems, **do not** use your pen without help from a person trained to use the Mounjaro pen.

## Where to learn more

- If you have questions or problems with your Mounjaro pen, contact your local Lilly affiliate or your doctor, pharmacist or nurse.

For any information about this medicine, please contact the local representative of the Marketing Authorisation Holder:

**United Kingdom (Great Britain)**  
Eli Lilly and Company Limited  
Tel: + 44-(0) 1256 315000

**Last revised in October 2023.**


12 of 13

BREAK  
SEAL

## Instructions for use

BREAK  
SEAL

**Mounjaro® 2.5 mg solution for injection in pre-filled pen**  
**Mounjaro® 5 mg solution for injection in pre-filled pen**  
**Mounjaro® 7.5 mg solution for injection in pre-filled pen**  
**Mounjaro® 10 mg solution for injection in pre-filled pen**  
**Mounjaro® 12.5 mg solution for injection in pre-filled pen**  
**Mounjaro® 15 mg solution for injection in pre-filled pen**  
tirzepatide

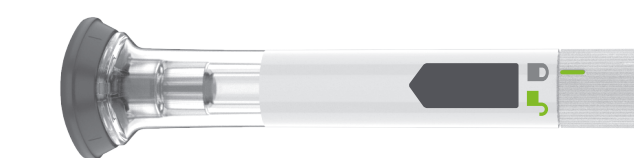

*Lilly*

13 of 13

NL024ITAA00

